
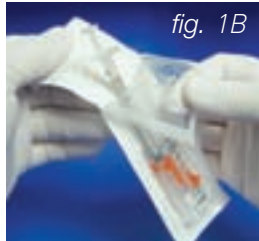


## Step-by-Step Guidelines


# Hypodermic Needle-Pro® Safety Device

**IMPORTANT:** Please refer to the instructions for use supplied with the product for detailed instructions, warnings, and cautions.

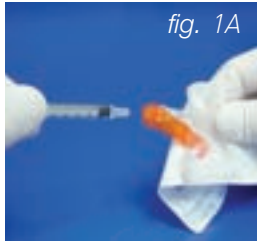

- 1 Needle with Needle Protection Device**  
Peel the blister pouch open halfway and grasp the needle cap using the pouch. To prevent contamination, be careful not to touch the needle hub. With an easy twisting motion, attach a Luer slip syringe to the needle hub (fig. 1A).

**Needle with Preattached Syringe and Needle Protection Device**  
Peel the blister pouch open and remove the entire device while grasping the syringe barrel. Position the Needle-Pro® device away from the needle. Seat the needle firmly on the Needle-Pro® device with a push and a clockwise twist (fig. 1B).

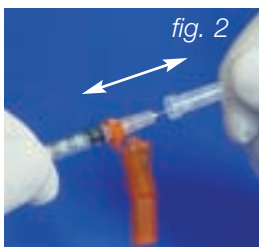

- 2** Remove the needle cap from the needle using a "push-pull" technique (fig. 2).

Draw up medication. Express air and administer medication.



- 3** Engage the Needle-Pro® device using a one-handed technique by gently pressing the sheath of the orange needle protection device against any flat surface. DO NOT use your free hand to press the protective sheath over the needle. As the sheath is pressed, the needle firmly engages into the sheath with an audible click (fig. 3A and 3B). Visually confirm that the needle is fully engaged into the needle protection sheath. Dispose of the entire device in an approved sharps container immediately after engaging the safety feature (fig. 5B).



### Smiths Medical ASD, Inc.

Norwell, MA 02061, USA

Phone: 1-214-618-0218

Toll-Free USA: 1-800-258-5361

The Smiths Medical and JELCO design marks, Needle-Pro are trademarks of Smiths Medical. The symbol ® indicates the trademark is registered in the U.S. Patent and Trademark Office and certain other countries. © 2011 Smiths Medical. All rights reserved. SS195225EN 04.2011

